

CEREC SW 4.5.2 material pack description

This material pack requires a CEREC SW 4.5.2 software license, which must be activated via the license manager. You can obtain the “CEREC_4.5.2_MP_Setup.exe” material pack either as an automatic update or you can download it manually from <https://my.cerec.com/en/products/cerec-sw.html>.

New material

- Manufactured by *Ivoclar Vivadent*, material: *Tetric CAD*, blocks I 12 and C 14; worldwide release
- Manufactured by *GC*, material: *CERASMART300*, blocks 12, 14 and 16; released only in Japan

Additional release in the USA


- Manufactured by *Kuraray Noritake Dental*, material: *KATANA Zirconia* block(ST)
- Manufactured by *Kuraray Noritake Dental*, material: *KATANA Zirconia* block (STML)

Name change

- A distinction is made between pre-integrated block sizes *Telio CAD A16 (S)* and *A 16 (L)* (manufactured by *Ivoclar Vivadent*) by renaming the block versions in relevant processes to *Telio CAD Abutment*

Installation

- Go to <https://my.cerec.com/de-de.html>, download the file “CEREC_4.5.2_MP_Setup.exe” and save it on your computer.
- Double-click the downloaded file to open it and select “Run”.
- Then accept the license agreement and select “Install”.

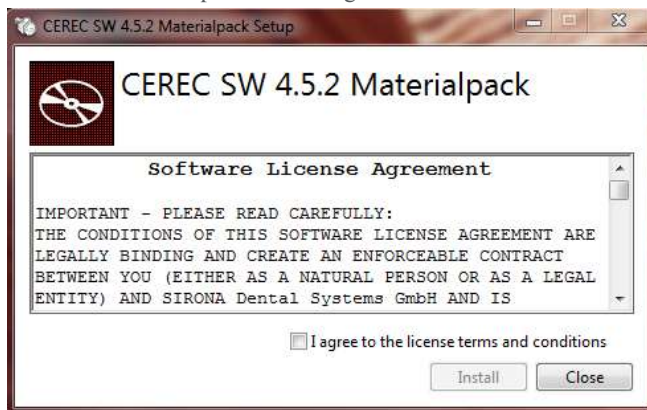

- The material pack is automatically installed, and the materials will be in the list the next time you start up your CEREC software.

- You can check whether the block database is correct by clicking on the CEREC software control panel. The correct version of the “block DB” is 452.1.368.

