
High Speed Handpiece
Fiber Optic / Non-Fiber Optic
 (Tradition® REF 750045 / 750044) (Quiet-air® REF 730014 / 464004)
 (Tradition-L® REF 780045 / 780044) (Quiet-air-L® REF 730014 / 464004)
 (Tradition-PB® REF 790045 / 790044)

Installation, Maintenance and Operation Instructions

Tradition® / Quiet-air® High Speed Handpieces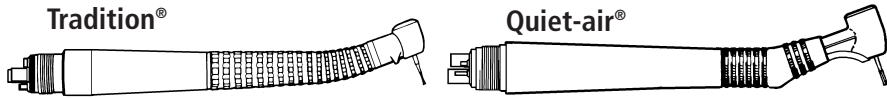

Indications For Use

This air-powered high speed handpiece is intended to be used for general dentistry procedures.

Contraindications

None Known.

Warnings

STERILIZE AND LUBRICATE BEFORE USE, SUPPLIED NON-STERILE.

WARNING: This product is intended for use by dental professionals only. Before operating handpiece, carefully read and follow these instructions and save them for future reference. Observe all cautions and warnings, including:

- Handpieces should be heat-sterilized between patients to prevent cross-contamination which could result in serious illness or even death from infectious organisms such as HIV (the virus that causes AIDS) or hepatitis B.
- It is the responsibility of the Dental Healthcare Professional to determine the appropriate use of this product and to understand the health of each patient, the dental procedures being undertaken and industry and governmental agency regulations, recommendations and requirements for safe practice of dentistry.

Precautions

- Do not cut with the handpiece unless you are sure the bur is secure. Do not extend the bur outward from the handpiece head. A loose or extended bur could eject from the head or break and cause personal injury.
- Do not use this handpiece within the oral cavity when a soft tissue wound is present. Air pressure may force septic materials into these tissues.
- Midwest turbines should not be rebuilt. Replacing bearings only could result in unexpected failure and patient injury due to different wear factors of new vs. old components.


Adverse Reactions

None Known.

Installation

Checking Handpiece Hose

Make sure all tubings supplying the handpiece hose are properly connected within the dental unit. Be sure the air and water are delivered to the handpiece as shown in the illustration below.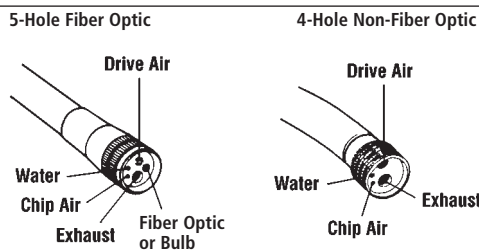

Air Requirements

The handpiece delivers designed speed and torque when operated with clean, filtered, moisture-free air at the recommended operating pressure of 30 p.s.i. (2.0 Atm.). Air pressure should be measured at the handpiece end of the hose.

Water Supply Recommendations

It is highly recommended that all dental water supply systems conform to applicable CDC (Centers for Disease Control and Prevention) and ADA (American Dental Association) standards, and that all recommendations be followed in terms of flushing, chemical flushing and general infection control procedures.

Initial Maintenance

Follow the **Maintenance** procedure, below, before using your new Tradition or Quiet-air handpiece.

Maintenance

Important: Only use Midwest® products recommended by DENTSPLY to routinely maintain your handpiece. See **Parts** section for ordering information.


There are two **Maintenance** procedures for your Tradition or Quiet-air handpiece: (1) for maintenance using the Midwest Plus™ Maintenance Kit (REF 380145) ; (2) for maintenance using the Midwest Plus™ Aerosol Spray (REF 380080). Follow the appropriate maintenance procedure, at right, before using your new Tradition or Quiet-air handpiece.

These procedures recommend the steps to follow between patient use to prevent cross-contamination and to ensure continued trouble-free operation of the handpiece:

1 Maintenance Using Midwest Plus™ Maintenance Kit

Clean

1. Remove bur.
2. Spray Midwest Plus™ Handpiece Cleaner liberally: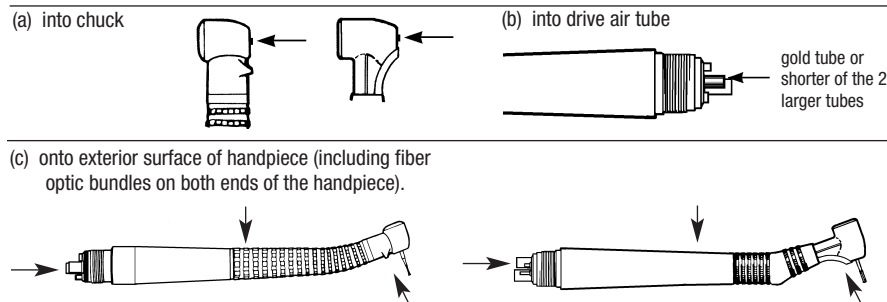

Scrub and Rinse

Scrub and rinse under warm tap water. Make sure fiber optic bundles are scrubbed well.

Expel Cleaner and Dry

Run handpiece on Midwest Handpiece Air Station (REF 380800) or other air source for 30 seconds or until handpiece reaches normal speed, and all excess cleaner is expelled. Handpieces with bur tool chucking system (Tradition REF 750045/750044 or Quiet-air REF 730014/464004) should always be covered with a Spray Guard or have a bur installed before running. Dry thoroughly.

Lubricate and Expel


1. Lubricate by placing 3 drops of Midwest Plus Handpiece Lubricant (REF 380130) into drive air tube (shorter of two larger tubes).
2. Run handpiece for 30 seconds or until it reaches normal speed, and until all excess lubricant is expelled. Wipe off any excess lubricant on outside of handpiece with a dry gauze pad or tissue.

2 Maintenance Using Midwest Plus™ Aerosol Spray

Scrub and Rinse

Remove bur. Scrub and rinse under warm tap water. Make sure fiber optic bundles are scrubbed well.

Clean, Lubricate and Expel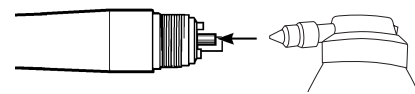

Clean and lubricate with one quick spray of Midwest Plus™ Aerosol Spray (REF 380080) using the Midwest Backend Attachment nozzle (REF 380086) into the drive air tube of the handpiece. The drive air tube is the shorter of the two larger tubes and is gold on many handpieces. Run the handpiece on Midwest Handpiece Air Station or other air source for 30 seconds or until normal speed is reached, and all excess spray is expelled. This handpiece with a bur type chucking system should always be covered with a Midwest Spray Guard or have a bur installed before running. Dry thoroughly.

Bag • Autoclave

Insert handpiece into sterilization bag and load into autoclave. (DENTSPLY recommends using sterilization bags with at least one paper side.) Handpieces should be processed in an autoclave set for a minimum of 15 minutes at sterilization temperature (132°C/270°F).

For cassette chamber saturated steam devices, handpieces should be processed unbagged for a minimum of 3.5 minutes at sterilization temperature (135°C/275°F).

Keep handpiece in bag until ready to use in the operatory. No relubrication is necessary.

Other Maintenance


Ultrasonically Clean Handpiece Head

As needed, only if sluggishness occurs:

1. Place handpiece head down in beaker of Midwest Plus Handpiece Cleaner.
2. Ultrasonically clean for 10 minutes.
3. Continue from **Scrub and Rinse**, above.
4. In the lubricating step, also place 1 drop of Midwest Plus Handpiece Lubricant (REF 380130) in chuck (for Push Button and Latch models: REF 790045 / 790044 / 780045 / 780044 / 780014 / 484004).

Cleaning Water Spray Tube

Run the cleaning wire back and forth inside the inner water spray tube and around outer chip air tube to loosen and remove mineral build-up at spray tip.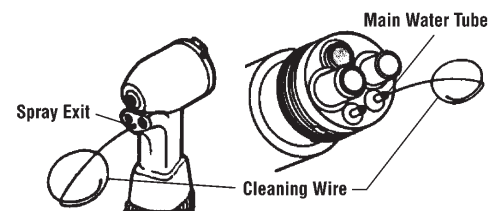

Operation (Standard: REF 750045 / 750044 / 730014 / 464004)

Never operate the handpiece under the following conditions:

1. In the presence of a soft tissue wound.
2. With a bur extended out from handpiece head.
3. Under "no load" conditions for extended periods of time.
4. At air pressure exceeding 30 p.s.i. (2.0 Atm.).

To operate the handpiece: Simply depress the foot control on your dental unit. For water spray and chip air, refer to dental unit instructions.

Use of the bur wrench is required to open and close the chuck when installing and removing burs.

Installing and Removing Burs: Tradition (REF 750045 / 750044)

Tradition handpieces accept regular friction grip (FG) burs and short shank (SS) burs.

To use bur wrench: Insert square pin of bur wrench through the handpiece's head cap until it engages the square socket in the chuck. Fit the two wrench pins into the matching slots in the head cap. Turn the wrench **counterclockwise** to open the chuck, **clockwise** to tighten.


To install a bur: Open chuck, insert bur all the way into the chuck. Tighten chuck with bur wrench.
Make sure bur is secure before operating handpiece. Always use bur wrench to tighten chuck.

To remove bur: Insert the bur wrench into the back of the handpiece head. Rotate the wrench **1/2 turn counterclockwise**, then remove bur.

Note: The water spray will adequately cover bur and diamond cutting instruments up to .906 inches (23 mm) long.

See Maximum Recommended Bur Length and Warning Information On Reverse Side.

Bur Wrench Tradition: (Wrench REF 750119R)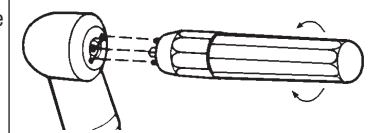

Operation (Standard: REF 750045 / 750044 / 730014 / 464004) continued

Installing and Removing Burs: Quiet-air (REF 730014/464004)

Quiet-air handpieces accept regular friction grip (FG) burs.

To use bur wrench: Insert square pin of bur wrench through large opening in stator cap until it engages square socket in the chuck. Fit the round wrench pins into two of the three matching corner slots in the stator cap. Turn upper knurled section of the wrench counterclockwise to open the chuck, clockwise to tighten.

Do not tighten chuck without bur installed.

Do not operate handpiece without bur installed and properly secured in chuck. These conditions could possibly damage chuck and cause bur ejection and/or chuck ejection. Bur or chuck ejection could cause patient injury.


To install a bur: Open chuck. Insert bur all the way into the chuck. If taper of bur drops below top of head, pull it out slightly, as shown, and tighten chuck. **Do not extend bur any further (read warnings).** Make sure bur is secure before operating handpiece.

Note: Tightening chuck with bur taper buried in the head will damage the chuck, resulting in a loss of concentricity and gripping ability.


Note: Use of oral surgical length burs could possibly result in handpiece bearing failure or personal injury due to bur ejection, bent burs, or inadequate cooling.

To remove bur: Insert the bur wrench into stator cap. Give knurled portion of wrench 1/2 turn counterclockwise, then remove bur.

Bur Wrench Quiet-air: (Wrench REF 463102R)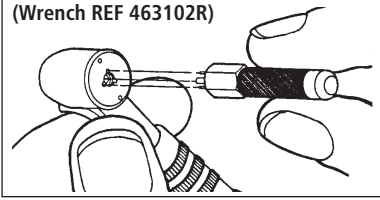

Bur Taper

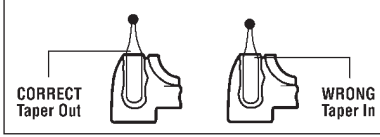
Maximum Recommended Bur Length (REF 750045 / 750044 / 790045 / 790044)

Note: Use of cutting instruments over .906 inches (23 mm) in length could possibly result in handpiece bearing failure or personal injury due to bur ejection, bent burs, or inadequate cooling.

Note: Do not extend burs outward from handpiece head. Burs must be inserted all the way into chuck for proper bur retention. Extended burs could result in personal injury due to bur ejection, bent burs or inadequate cooling.

Maximum Recommended Bur Length

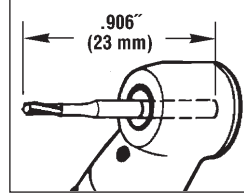Repair

Midwest Air Repair™ Factory Certified Service

Should your Midwest Quiet-air High Speed Handpiece need repair, call 1-800-800-7202 to schedule a free pick-up:

Send to:
DENTSPLY Professional
Attn: Midwest Air Repair
901 West Oakton Street
Des Plaines, IL 60018-1884

Provide the following information for your repair:

- Complete name and address
- Dealer name and address
- Nature of repair

Air Repair offers:

- Repairs DOOR TO DOOR IN 2 BUSINESS DAYS
- Genuine Midwest Parts and Midwest Quality
- Expert Troubleshooting and Diagnosis
- Repair Service Warranty equal to the new product warranty

Operation (Push Button: REF 790045 / 790044)

Never operate the handpiece under the following conditions:

1. In the presence of a soft tissue wound.
2. With a bur extended out from handpiece head.
3. Under "no load" conditions for extended periods of time.
4. At air pressure exceeding 35 p.s.i. (2.4 Atm.).
5. With a loose or bent bur.

To operate the handpiece: Simply depress the foot control on your dental unit. For water spray and chip air, refer to dental unit instructions.

Push Button

The Push Button Chucking System of your new Tradition•PB handpiece accepts regular friction grip (FG) burs and short shank (SS) burs.

Installing and Removing Burs

To install a bur: Insert the bur into the handpiece until you meet resistance. Now press the button and push the bur all the way into the chuck until it stops.

Make sure bur is secure before operating handpiece.

To remove bur: Push the button to open chuck; then remove bur.

Note: The water spray will adequately cover bur and diamond cutting instruments up to .906 inches (23 mm) long.

See Maximum Recommended Bur Length and Warning Information At Top Of Next Column.

Operation (Power Lever®: REF 780045 / 780044 / 780014 / 484004)

Never operate the handpiece under the following conditions:


1. In the presence of a soft tissue wound.
2. While lifting Power Lever.
3. With a bur extended out from handpiece head.
4. Under "no load" conditions for extended periods of time.
5. At air pressure exceeding 30 p.s.i. (2.0 Atm.).

To operate the handpiece: Simply depress the foot control on your dental unit. For water spray and chip air, refer to dental unit instructions.

Power Lever®

The Power Lever Chucking System of your new Tradition•L handpiece accepts regular friction grip (FG) burs and short shank (SS) burs. The Power Lever Chucking System of your new Quiet-air•L handpiece accepts only regular friction grip (FG) burs. No bur wrench is required — simply lift lever up to open chuck; then flip it down to lock the bur in place.

Note: When lever is in up position, the chuck is open and the turbine cartridge will not rotate.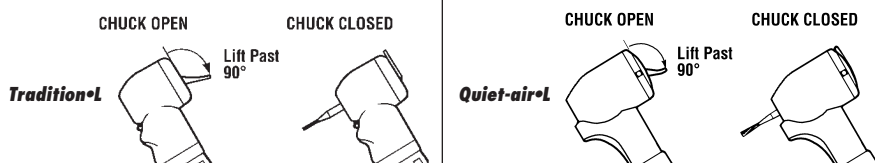

Installing and Removing Burs

To install a bur: Lift up the lever — past 90° position — to open the chuck. Insert bur shank all the way into the chuck; then flip Power Lever down to lock bur in place.

Make sure bur is secure before operating handpiece.

To remove bur: Lift lever (past 90°) to open chuck; then remove bur. When the handpiece is not in use, store it with a bur installed, lever down.


Note: The water spray will adequately cover bur and diamond cutting instruments up to .906 inches (23 mm) long.

See Maximum Recommended Bur Length and Warning Information At Top Of Next Column.

Adjusting Power Lever Position

With your fingers (or the side of your thumb) push the Power Lever around to the position desired. Lever turret may be rotated in either direction.

Important: Power Lever cannot be rotated while it is in the up (opened) position.

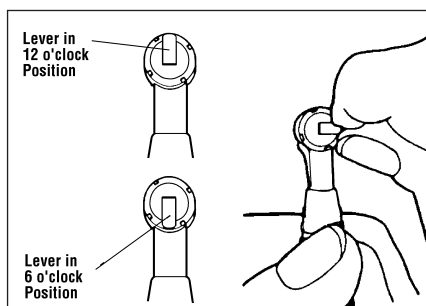Parts

REF

Midwest Plus Aerosol Spray	Cleaner/Lubricant (does not include nozzle)	380080
Nozzle for Midwest Plus Aerosol Spray		380086
Midwest Plus Maintenance Kit	8 oz. Spray Cleaner & 2 oz. Lubricant Dropper	380145
Midwest Plus Handpiece Cleaner	8 oz. Spray Bottle 1 Liter Refill Bottle	380140 380141
Midwest Plus Handpiece Lubricant	2 oz. Dropper 8 oz. Refill Bottle	380130 380131
Midwest Handpiece Air Station	includes box of 12 FREE Spray Guards	380800
Midwest Spray Guards	Box of 100	463102
Spray Tips	for Midwest Plus 8 oz. Spray Cleaner, pkg of 6	380986
Tradition Turbine Cartridge Replacement Kit	with cap wrench	750118
Tradition•L Turbine Cartridge Replacement Kit	less cap assembly	780118
Tradition•PB Turbine Cartridge Replacement Kit	with cap wrench less cap assembly	790118
Tradition Bur Wrench		750119R
Quiet-air Turbine Cartridge Replacement Kit	with cap wrench	464300
Quiet-air•L Turbine Cartridge Replacement Kit	less cap assembly	484300
Quiet-air Bur Wrench		463102R
Handpiece Gasket	4-tube Midwest 5-tube Fiber Optic For Use With Q-Hose: 5-tube Fiber Optic, red	773002R 773018R 753342R
Cleaning Wire Kit		464197R
Tradition•L Head Cap	Replacement cap	780002R
Quiet-air•L Head Cap	Replacement cap	484301R

Warranty

Your Tradition / Tradition•L / Tradition•PB and/or Quiet-air / Quiet-air•L Handpiece is guaranteed to be free of defects in materials and workmanship for a period of six (6) months from its date of purchase — provided that the handpiece has been operated and maintained as prescribed in these instructions (at the discretion of DENTSPLY Professional) and has not been subjected to apparent misuse, abuse or accident. All questionable, defective, or loose parts must be returned to the dealer, with handpiece and proof of purchase, transportation prepaid.

The enclosed business reply card (guaranty registration) must be filled out completely and returned within ten (10) days of purchase for this guaranty to apply. DENTSPLY Professional makes no other guaranties, expressed or implied.

MIDWEST
AIR REPAIR
FACTORY CERTIFIED SERVICE

For the Dentist and Staff:

Handpiece Infection Control and Maintenance Information

Information on your Midwest Repair

Handpiece Problem Solving

Phone: **1-800-800-7202**

DENTSPLY
PROFESSIONAL

DENTSPLY Professional
A Division of DENTSPLY International
901 West Oakton Street
Des Plaines, IL 60018-1884 USA

Copyright 2008 DENTSPLY International
Printed in U.S.A. Code No. 87074-0708
U.S. Patent No. 4330279.

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