Dentsply Sirona

Dispersalloy®

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- Work in ventilated rooms with decontaminable non-textile surfaces.
- Realize shaping and polishing of amalgam always under cooling and suction of the operation field.
- Condense amalgam with classical methods (plugger) and do not use ultrasound condensers
- Do not place amalgam in direct proximity of other metallic restorations in order to avoid all risk of corrosion.
- Avoid placing and removing amalgam during pregnancy and lactation.
- If elated reactions, in particular lichoil lesions in the proximity of amalgam, occur, it is justified to remove the filling.

2.3 Adverse reactions (When mixed with mercury as intended)

- Product may irritate skin, eyes. Skin contact: irritation or possible allergic response. Reddish rashes may be seen on the skin. Eye contact: Irritation and possible corneal damage. (See Warnings and Precautions)
- Inhalation of vapors may cause varying degrees of damage to the affected tissue and also increased susceptibility to respiratory illness. (See Warnings and Precautions)
- Product may cause neurotoxic, nephrotoxic or other serious health effects if inhaled or ingested. (See Warnings)

2.4 Storage

Dispersalloy Alloy powder or tablets should be stored in a well ventilated place at temperature not to exceed 25°C/77°F. Do not allow exposure to moisture or excessive humidity. Do not use after expiration date.

3. STEP-BY-STEP INSTRUCTIONS

- Treatment of the preparation: Clean freshly instrumented enamel dentin with water spray and then air dry. The use of pulpal protection, bases and/or cavity liners is at the discretion of the operator, as dictated by the clinical presentation. If desired, Dispersalloy Alloy may be placed over an adhesive liner or adhesively bonded by following the instructions contained in the compatible Adhesive System (available separately).
- Proportions: As with all amalgam alloys, the ratio of mercury to alloy is important. When using either Dispersalloy Tablets or Powder, a 1:1 mercury:alloy ratio is recommended. Use of an automatic powder and mercury proportioner is strongly recommended. It will automatically and precisely dispense measured amounts of mercury and alloy. Refer to the instructions provided by the proportioner manufacturer.
- Trituration: Trituration speed and time is an important factor in preparing the amalgam, in order to achieve proper amalgam mix consistency. Use of the DENTISPLY Rinn® Crescent® Wig-L-Bug® Screw Type Capsule and Pestle is recommended. The chart that follows provides guidelines in determining amalgamotr trituration time and speed setting for Dispersalloy Alloy:

Approximate Trituration Time Range (in seconds)							
Fast Set							
Amalgamator	Setting	1 Tablet	2 Tablets	1 Tablet	2 Tablets		
Cauk ProMix®	Turtle	14x3	14x3	14x3	12x3		
Cauk Vari-Mix IIM	M2	14x3	14x3	14x3	12x3		
Cauk Vari-Mix III	M2	14x3	14x3	14x3	12x3		
ESPE CapMix®	--	5x1	5x1	NR	NR		
Wig-L-Bug D580	High	13x2	13x2	13x2	9x2		
Wig-L-Bug MSD 3800	3800	16x3	16x3	14x3	12x3		
ADEC®	5	14x3	14x3	14x3	12x3		
Silamat®	4200	5x1	5x1	NR	NR		
Silamat Plus®	P-2 Slow	8x2	6x2	6x2	NR		
Powder							
Amalgamator	Setting	400mg	600mg	800mg	400mg	600mg	800mg
Cauk ProMix®	Turtle	14x3	14x3	12x2	14x3	14x3	12x2
Cauk Vari-Mix IIM	M2	14x3	14x3	12x2	14x3	14x3	12x2
Cauk Vari-Mix III	M2	14x3	14x3	12x2	14x3	14x3	12x2
ESPE CapMix®	--	5x1	5x1	NR	NR	NR	NR
Wig-L-Bug D580	High	14x3	14x3	12x2	14x3	14x3	12x2
Wig-L-Bug MSD 3800	3800	14x3	14x3	12x2	14x3	14x3	12x2
ADEC®	5	14x3	14x3	14x3	14x3	14x3	12x2
Silamat®	4200	5x1	5x1	5x1	5x1	5x1	5x1
Silamat Plus®	P-2 Slow	7x1	6x1	6x1	7x1	6x1	6x1
*Not products of DENTISPLY International							

- The triturated amalgam should be bright and have a plastic consistency. If the amalgam is excessively splashy and wet looking, INCREASE trituration time or amalgamator speed. Amalgam that appears excessively dry indicates trituration time or amalgamator speed should be DECREASED.
 - Suggestions for obtaining optimum mix: When performing adjustments, it is recommended to adjust trituration time in 2 second increments, until optimum mix is obtained. Greater adjustments may indicate amalgamotr incompatibility or compromised functionality.
- Decrease trituration time Increase trituration time
- to lengthen working time. - to shorten working time.
 - if mix is too hot. - if mix is too wet.
 - if mix is too dry.
- *Properly triturated Dispersalloy® Amalgam does not require mulling to produce a smooth plasticity; however you may mull for 1-2 seconds after removing pestle to ensure amalgam into a single mass.
4. Condensation: Dispersalloy® Alloy is now ready for immediate placement and condensation. A gentle tap may be required to free amalgam from the capsule. Do not attempt to manually express mercury from the triturated mass prior to placement.
- Condense immediately after mix is completed. Use a clean amalgam carrier to transfer increments to the cavity. Pack each increment into angles and undercut with small-faced plugger and sufficient pressure to insure good adaptation. Build the restoration with additional portions, until the cavity is slightly overfilled. Remove any mercury-rich amalgam from the surface that may develop during condensation.
5. Carving/Burnishing/Carving can begin immediately after condensation. A sharp carver will give best results. The anatomy should be carved proceeding from the proximal margins towards the center of the restoration. Before the matrix band is removed, the marginal ridge should be supported by a condenser. After removal of the matrix band, gingival margins may be smoothed with a curved explorer. Burnishing setting amalgam is recommended to improve surface smoothness.
6. Polishing: The quality of the Dispersalloy Alloy restoration will be enhanced by polishing. The amalgam should be allowed to set 24 to 48 hours before polishing. See selected polishing instrument manufacturer's directions for use.

4. HYGIENE

- Properly dispose of the amalgam scrap in accordance with local regulations.

5. LOT NUMBER AND EXPIRATION DATE

- Do not use after expiration date. ISO standard is used: "YYYY/MM"
- The following numbers should be quoted in all correspondence.
 - Rearder number
 - Lot number on package
 - Expiration date on package

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