Maintenance Guide

IMPORTANT: Turn OFF water supply to Cavitron system or disconnect water quick-disconnect from the water supply when not in use.

Start-Up Procedures (beginning of day):

1. Open the manual shut-off valve on the dental office water supply system.
2. Install a sterilized Steri-Mate Handpiece onto the handpiece cable.
3. Set the Power Level Control to minimum and the Lavage Control to maximum.
4. Turn the system ON.
5. Hold the sterilized handpiece (without insert installed) over a sink or drain. Activate the Purge Mode by pressing the Purge icon.
 - The Purge screen will appear indicating proper activation of the purge function. Time icon will count down indicating completion of purge cycle.
 - The Purge function can be interrupted at any time during the two minute cycle by pressing the Scale icon or by pressing the Foot Pedal.
6. After completing the purge cycle, place a sterilized 30kHz Cavitron Ultrasonic Insert into the handpiece and set the Power Level Control and Lavage Control to your preferred operating position for ultrasonic scaling.

Between Patients:

1. Remove the used Cavitron Ultrasonic Insert. Clean and sterilize following the Infection Control Procedures that were enclosed with your insert.
2. Hold the handpiece over a sink or drain and activate Purge function as described in Step 5 of the Start-Up procedure.
3. After the Purge cycle is complete, turn the System to the OFF (O) position.
4. Remove the Steri-Mate handpiece, clean and sterilize following the procedures outlined in the Cavitron Systems Infection Control Procedures booklet that was enclosed with your system.
5. Before disinfecting surface of cabinet, lock the touch screen by pressing the Lock icon. Disinfect the surfaces of the cabinet, Power Cord, Handpiece Cable, Tap-On Foot Pedal and Cable assembly (if applicable), Water Supply lines as instructed in the Cavitron Systems Infection Control Procedures booklet. **DO NOT SPRAY DISINFECTANT SOLUTION DIRECTLY ON SYSTEM SURFACES*.**
6. Inspect the handpiece cable for any breaks or tears.
7. If using a closed water supply or DualSelect Dispensing System, check for adequate fluid volumes for the next patient.
8. When ready to use, press the unlock icon, place a sterilized Steri-Mate Handpiece onto the handpiece cable assembly and insert a sterilized ultrasonic insert into the handpiece and adjust system controls as preferred.

Shut-Down Procedures (end of day):

Follow the "Between Patients" maintenance procedures, Steps 1 through 6. If a quick-disconnect is attached to the end of the hose:

1. Disconnect the water supply line from the dental office water supply.
2. Relieve the water pressure by pressing the tip of the connector into an appropriate container and allow water to drain.

If quick-disconnect is not attached to the end of the hose, it is recommended to close the manual shut-off valve on the dental water supply system

*NOTE: Refer to the Infection Control Procedure Booklet supplied with your system for disinfectant products that have been tested for compatibility.

Extended periods of closure:

Refer to "Shut-Down Procedures (end of day)" maintenance, in addition:

1. Remove the hose from the system, by pushing on the outer ring of the system's water inlet and gently pull out the waterline.
2. As an additional precaution, blowing air through the waterline (using an air/water syringe) may also be helpful when preparing for extended inactive use of a Cavitron Ultrasonic Scaling System

Handpiece Cable

While the Cavitron Handpiece Cables are designed to offer optimum flexibility and endurance, improper handling may lead to a shortened useful life of the cable assembly, due to excessive wear on the wires and/or internal tubing. To prevent premature failure of the cable assembly, please use the following steps:

1. Do not wrap the cable tightly around the Cavitron system or any apparatus.
2. Avoid overstretching or pulling the cable.
3. If possible, minimize frequent bending/folding that exceeds normal usage during dental procedure.
4. For detachable handpiece cables, hold by the plug body when detaching from the Cavitron system.
5. Gently loop the cable in a continuous loose circle after each use.

Waterline Maintenance Instructions

Dental Unit Waterline Overview:

Dental Unit Waterline Maintenance is a key component to a facility's effective Infection Control and Prevention Protocol. The Centers for Disease Control and Prevention (CDC) recommends that dental unit water used in nonsurgical procedures measure less than or equal to 500 colony forming units of heterotrophic bacteria per milliliter (≤ 500 CFU/mL) of water, the standard set for drinking water by the Environmental Protection Agency (EPA). All dental unit waterlines should be treated, shocked, and tested following CDC recommended guidance. If outside the US, standards may differ, check your local regulations on safe drinking water.

Before Use

Dentsply Sirona recommends that you perform waterline testing on your new Cavitron scaling system prior to use on a patient.

INDEPENDENT WATER BOTTLE/RESERVOIR CONNECTION
<p>If connecting a Cavitron Scaling System waterline to a dental chair, where the water is delivered via an independent water bottle / reservoir, follow the dental chair manufacturer's Instructions for Use for treating and shocking your waterlines.</p>
<p>1. The waterline maintenance products listed below have been tested for material compatibility with Cavitron Ultrasonic Scaling Systems:</p>
<p><u>Waterline Treatment Products</u></p>
<p>a-dec ICX™ Tablet Waterline Treatment Hu-Friedy Waterline filter ProEdge Dental Products BluTab® waterline tablets Crosstex DentaPure® cartridge Sterisile CITRISIL™ Dental Waterline Microbial Contaminant Control Tablet Alpro Alpron Liquid Concentrate Dentsply Sirona Dentosept® S water disinfectant.</p>
<p><u>Waterline Shock Products</u></p>
<p>Crosstex DentaPure® cartridge</p>
<p>2. Follow manufacturer instructions for use for the waterline treatment products listed above.</p>
<p>NOTE: Ensure that Cavitron Scaling System is connected to the dental chair when performing the shock procedure recommended in the chair manufacturer's IFU.</p>
MUNICIPAL WATER CONNECTION
<p>If connecting a Cavitron Scaling System waterline directly to a Municipal water valve, treat the waterlines weekly with a 1:10 Sodium Hypochlorite 5.25% Solution with the following steps.</p> <ol style="list-style-type: none">1. Connect the Cavitron Scaling System to the Cavitron DualSelect Dispensing System following the DualSelect system's Instructions for Use.2. Prepare a 1:10 Sodium Hypochlorite Solution by adding a 5.25% Sodium Hypochlorite to the designated line on the orange disinfectant bottle.3. Fill remainder of disinfectant bottle with tap water4. Place the orange disinfectant bottle in position "A".5. Close the shield and set the Selector Knob to position "A"6. Adjust the lavage/water control at the base of the handpiece to the maximum water flow and begin to flush the 1:10 Sodium Hypochlorite Solution through the system for at least 30 seconds, ensure that the solution is exiting the handpiece.7. Set the Selector Knob to Open H₂O position and open the shield.8. Remove the orange disinfectant bottle from position "A" and place it in position "B" and close the shield. Switch selector knob to position "B" and operate the unit for at least 10 seconds.9. Set selector knob to Open H₂O position, open the shield, and remove the disinfectant bottle.10. The system should be left undisturbed, allowing the prepared solution to sit in the waterline, for a minimum of 10 minutes but no more than 30 minutes.11. As a precaution, place a sign on the scaling system indicating it is being treated and should not be used12. After the minimum 10-minute wait time has passed, fill another bottle with clean water, place it in position "A" and close the shield. Set the Selector Knob to position "A" and operate the Cavitron System at maximum waterflow for at least 2 minutes or until all Sodium Hypochlorite odor disappears.13. Set selector knob to Open H₂O position, open the shield. Remove the clean water bottle from position "A" and place it in position "B" . Close the shield and set selector knob to "B". Once again, operate the Cavitron system at maximum waterflow for at least 2 minutes, until all the water drains and/or all the Sodium Hypochlorite odor disappears.14. Ensure that all the Sodium Hypochlorite solution has been completely flushed from the system prior to use on a patient.15. Set the selector Knob to "Open H₂O", open the shield and remove the empty bottle from position "B".16. Place empty bottle in position "A", close the shield and set selector knob to position "A".17. Operate Cavitron System at maximum waterflow for at least 30 seconds to air purge line dry.18. Set selector knob to "open H₂O" position, open shield and remove empty bottle for position "A" and place in position "B".19. Operate Cavitron System at maximum waterflow for at least 30 seconds to air purge line dry.
<p>NOTE - ALL Sodium Hypochlorite solution MUST BE FLUSHED FROM THE SYSTEM BEFORE IT IS READY FOR PATIENT USE</p>

Water Filter Maintenance Guide

Your Cavitron system is equipped with a waterline filter that needs to be evaluated and replaced periodically.

WATER QUALITY:	MUNICIPAL	WELL	HARD	SOFT	BOTTLE SYSTEMS
Recommended Evaluation and/or Replacement:	1 x month	2 x month	2 x month	1 x month	2 x month
Other Considerations:					If using chemical cleaners, the system should be flushed appropriately to prevent damage to equipment. Please refer to the Cavitron Maintenance Guide for recommended procedure.

KEY INDICATORS THAT YOUR CAVITRON WATER FILTER NEEDS TO BE REPLACED:

- Water flow is reduced to the insert and/or handpiece
- If particulates are visible on the filter
- Discoloration of the filter
- Light indicator on your unit is visible

PLEASE NOTE:

- A 10-pack of replacement filters (part number 90158) is available from your local Dentsply Sirona distributor.
- The Cavitron water filter is not intended to be the primary water filtration for the office. It is meant to protect the Cavitron system and components.
- If your Cavitron water filter needs to be replaced on a highly frequent basis, you should consider an overall water filtration system for your office to minimize damage to your dental equipment.
- Please refer to the scheduled maintenance guide for your specific Cavitron system for complete maintenance recommendations.

TO REPLACE YOUR CAVITRON WATER FILTER:

1. Verify that the system is turned OFF.
2. Locate the blue waterline on the back of the unit.
3. Disconnect the water supply hose from the dental office water supply. If a quick disconnect connector is attached to the end of the hose, relieve the water pressure by pressing the tip of the connector in an appropriate container and drain the water.
4. Grasp the fittings on either side of the filter disk and twist counterclockwise. Remove the filter section from either side of the water hose.
5. Install the replacement filter onto the water hose fittings. The filter should be positioned to match up with the correct hose fitting.
6. Hand tighten one hose fitting onto filter in a clockwise direction. Tighten second hose onto filter in clockwise direction. Reconnect the water supply line, operate unit to bleed the air and test for leaks.

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